

## H&E Set-up for the Leica Autostainer (Program 1)

<u>Station</u>	<u>Reagent</u>	<u>Time</u>	<u>Exact or Not Exact</u> (Yes/NO)
1	Xylene	7 min.	No
2	Xylene	7 min.	No
3	Xylene	5 min.	No
4	Abs Etoh	2 min.	No
5	Abs Etoh	2 min.	No
6	95% Etoh	2 min.	No
7	70% Etoh	1 min.	No
Wash 5	Tap Water	1 min.	No
8	D H2O	20 sec.	No
9	Hematoxylin	4 min.	Yes
Wash 4	Tap Water	5 min.	No
	Ammonia H2O	1 min.	No
Wash 2	Tap Water	5 min.	No
10	70% Etoh	30 sec.	No
11	Eosin	2 min.	Yes
12	95% Etoh	30 sec.	Yes
13	95% Etoh	30 sec.	Yes
14	Abs Etoh	45 sec.	No
15	Abs Etoh	2 min.	No
16	Abs Etoh	1 min.	No
17	Xylene	2 min.	No
18	Xylene	2 min.	No
Exit	Xylene		